
ONLINE PACS ACCESS

Guilford Radiology provides free online, secure access to your patients reports and images through SECTRA UniView PACS

- Computer requirements for IDS7:
 - Microsoft Windows 7, 8 .1 or 10 (Windows 7 no longer supported)
 - Microsoft Internet Explorer 8, 9, 10 or 11 or Microsoft Edge
 - Microsoft.Net Framework 4.7.1 or later
 - A viewer for pdf documents, Adobe Reader version 10.1 or later is supported

- Installation and Login Instructions:
 - To install application:
 - ✓ Open Internet explorer
 - ✓ Go to www.guilfordradiology.com
 - ✓ Top right corner, click on " For Referring Physicians"
 - ✓ Middle of screen, click on "View images online – click here"
 - ✓ The first time you run Uniview you may need to install several plug-ins.

 - To Login:
 - ✓ Follow instructions above or
 - ✓ Open internet explorer and go to pacs.ramradiology.com/uniview
 - ✓ Enter Username:
 - ✓ Enter Password:
 - ✓ Search patient by name

Please call Guilford Radiology at 203-453-5123 if you need assistance or to change or reset your username/password.